



myglucohealth®

The New MyGlucoHealth® Wireless meter is the world's first integrated Bluetooth-enabled blood glucose meter. With the smallest blood sample size (0.3µL), automatic coding and results in just 3 seconds, the MyGlucoHealth Wireless meter is the most technically advanced diabetic care product on the market today.

The MyGlucoHealth Wireless meter along with the MyGlucoHealth Network offer a comprehensive diabetes management platform that gives patients more direct control over their care, while providing clear lines of communication with their clinician, family and caregivers.

MyGlucoHealth Wireless works in combination with a real-time online data collection network to upload and report patient blood glucose readings. Using the MyGlucoHealth Network at www.myglucohealth.net, (or regional sites throughout the world) patients can transmit, review and evaluate their daily readings while communicating those same results automatically to their clinician or caregiver. This allows clinicians to stay up-to-date on patients and intervene immediately if needed.


Data in the MyGlucoHealth Network is secure, accurate, gathered in real-time and accessible to both patients and clinicians 24 hours a day, seven days a week from any Internet-connected computer.

Entra Health Systems received clearance for sale of the MyGlucoHealth Wireless meter in the United States (FDA K081703). The Meter also has CE Mark in Europe (15054) and TGA approval in Australia (ARTG 161202).

Smarter Diabetes Management for a Digital LifestyleSM

MyGlucoHealth Wireless

- World's first FDA-cleared Bluetooth® blood glucose meter
- Smallest sample size: 0.3 microliters
- Fastest testing: 3 second results
- Auto-coding
- Alternate site testing
- Plasma-calibrated
- Add blood to test strips for up to one minute
- Easy-to-read display, large clear buttons
- Price of meter and strips comparable to non-connected meters.
- Test Range — 10-600 mg/dL (0.6 - 33.3 mmol/L)
- Power Source — 3 V (Alkaline Battery, 1.5V AAA Size X 2)
- Battery Life — 2,000 Tests

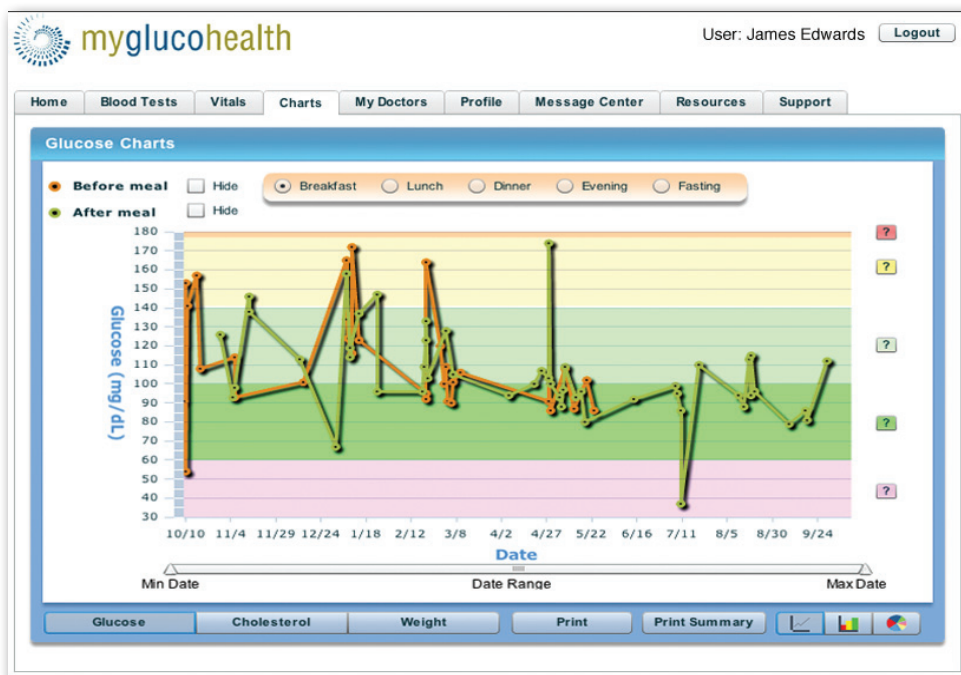

Make Better Lifestyle Choices

On the Patient Portal, readings are charted and interpreted in a number of ways to best analyze and evaluate blood glucose data. The Portal is designed to help each patient take "ownership" of their disease, so they can make better lifestyle choices.

Reminders, Messages & Alerts

The MyGlucoHealth Network allows patients to automatically send themselves or their caregivers alerts and reminders about their changing condition or to remind them to test or take medication. Alerts and reminders can be sent by email or SMS text message.

Each patient can create personalized high or low "threshold" settings for blood glucose level. The Network monitors uploaded results and automatically sends an alert to the patient, their family, physician or caregiver when the threshold is exceeded.

The MyGlucoHealth® Network



MyGlucoHealth® Wireless Meter Transmits Blood Glucose Test Data By Bluetooth to a Mobile Phone, PC or Macintosh



Data Is Uploaded to the Secure MyGlucoHealth Network Portal



MyGlucoHealth Network Portal



Real-time Access to Test Data for Patients, Families & Caregivers




Blood Glucose Log, Disease/Lifestyle Management & Historical Analysis Tools Available Online



Online Analytics & Patient Management Tools for Healthcare Professionals

Diabetes Management Product Comparison

Type of Meter	Manufacturers*	Description	Features	Advantages
Wireless Communicating Meters	MyGlucoHealth – Entra Health Systems 	A finger prick system that transmits results to a web-based secure patient portal using a mobile phone or PC via Bluetooth or using a USB cable to a PC or Macintosh. Price of meter and supplies comparable to non-connected meters.	<ul style="list-style-type: none"> • Upload results by mobile phone or PC using Bluetooth® • Smaller Blood Size 0.3µl • Faster readings - 3 seconds • Plasma calibrated for lab accuracy • Automatic coding of strips • Easy strip ejector • Alternate site testing • Large, easy to read display 	<ul style="list-style-type: none"> + Upload and review results on mobile phone using Bluetooth + Extensive patient analysis tools at MyGlucoHealth Network Portal online + Automated testing, medication reminders and alerts + Only FDA-cleared Solution
Traditional Meters	Freestyle – Abbott Diabetes Care Breeze – Bayer Contour – Bayer Lifescan – OneTouch from J & J Prestige IQ – Home Diagnostics Accu Check – Roche WaveSense - AgaMatrix, Inc	A finger prick system where users test themselves and manually record their results in a logbook. Some manufacturers offer PC-based software to record results.	<ul style="list-style-type: none"> • Testing times vary from 5-50 seconds • Blood sample size varies from 0.1 to 10 micro liters • Automatic and manual coding • PC-based electronic log book 	<ul style="list-style-type: none"> - Manually record results in a logbook or in PC log - Larger sample sizes - Longer testing times
Hybrid Communication Meter/Cradle/Hub Approach	T+ Mobile Phone Assistant – T-Medical Glucomon - Diabetech HealthPal - MedApps Fora - Fora Care Inc.	Meter and cradle solutions interconnect with existing name brand glucometers to transmit data to PC-based electronic log books or to online web applications.	<ul style="list-style-type: none"> • Separate product sold as an add-on to another glucometer • Testing time varies based on connected meter • Blood sample size varies based on connected meter 	<ul style="list-style-type: none"> - Cumbersome to manage and connect - May require subscription-based service - Information not available in Real-time

* All trademarks used are properties of their respective owners.



Americas
 Entra Health Systems LLC
 3111 Camino del Rio North, Suite 101
 San Diego, California 92108 U.S.A.
 Phone: 1 877 458 2646
 1 877 GLUCO 4 ME
 1 619 584 6704 (outside the US)
 Fax: 413 513 6343
www.myglucohealth.net
www.entrahealthsystems.com
 Email: support@myglucohealth.net

Australia and Asia
 Entra Health Systems Pty Ltd
 12/60 Castlereagh
 Sydney, NSW 2000 Australia
 Phone: 028 005 4745
 Fax: 028 088 7105

Europe, Middle East and Africa
 Entra Health Systems Ltd
 4A Blake Mews, Kew Gardens
 Surrey TW9 3GA United Kingdom
 Phone: +44 (0) 113 815 5108